

REMOVAL & INSTALLATION OF AMDENT SCALER TUBING

Handpiece Tubing Assembly
Amdent Module: A560-3841
Retail Price: \$350.00

(Tools needed: Phillips Screwdriver, Wire Cutters, Small Flat Screwdriver, Crescent Wrench)

REMOVAL

1. **REMOVE** the 4 PHILLIP SCREWS that hold the delivery system vertically to the formica top.
2. **REMOVE** the BLACK & WHITE WIRES at the junction box of the old handpiece tubing.
3. **REMOVE** the CLEAR WATER LINE by pulling back WHITE SLEEVE CLAMP. **CUT TUBING** from BARB.
4. **CUT** the FLUORESCENT SAFETY LINE, which **looks like yellow dental floss**.
5. **REMOVE** all ZIP TIES holding the tubing.
6. **REMOVE** TUBING by pulling through the BLACK RUBBER GROMMET on the delivery system.

INSTALLATION

1. **PUSH** TUBING UP through BLACK RUBBER GROMMET on delivery system.
2. **PULL** TUBING THROUGH HOLE to desired length or same length of tubing.
3. **TIGHTEN** ZIP TIE around TUBING just ABOVE BLACK RUBBER GROMMET.
4. **REMOVE** excess TUBING SHEATH (**1 INCH ABOVE**) ZIP TIE. Please be careful not to cut wires, waterline, or safety line.
5. **CUT** WATER LINE to proper length and install over barb.
6. **SLIDE** WHITE PLASTIC SLEEVE CLAMP onto WATER LINE.
7. **INSTALL** TUBING onto barb making sure to slide it on all the way.
8. **SLIDE** WHITE SLEEVE CLAMP down over barb.
9. **CUT** BLACK & WHITE WIRES to proper length, **LEAVING** about 4 **EXTRA INCHES**.

REMOVAL & INSTALLATION OF AMDENT SCALER TUBING

10. **STRIP BACK WIRING** to reveal 1/4 inch of BARE WIRE on both wires.
11. **TWIST WIRES** to create clean ends.
12. **INSTALL** into CLEAR ELECTRICAL BOX, which looks like a lego. Black to black.
White to white.
13. **TEST SCALER HANDPIECE** for function.
14. **REMOUNT DELIVERY SYSTEM.**
15. **RETEST SCALER HANDPIECE.**

If you have any questions before you attempt to install this, please call Paul or Daniel at **(800)866-6881**.