

REMOVAL & INSTALLATION OF WATER RELAY

Water Relay: A23-000

Retail Price: \$25.95

(Tools needed: Wire Cutters, Needle Nose Pliers, Phillips Screwdriver, Extracting Forcep)

REMOVAL

1. **REMOVE** the 4 **SCREWS** holding the delivery system to the wooden top.
2. **REMOVE** 1 **TUBING** at a time from the water relay. Please be careful not to mix them up.
3. **REMOVE** the **TUBINGS** by **PULLING AND TWISTING** on the **WHITE SLEEVE CLAMP**.

INSTALLATION

1. **SLIDE** a **WHITE SLEEVE CLAMP** UP the **TUBING**, then **SLIDE** the **WHITE CLAMP** DOWN OVER THE barb and tubing.
2. **REPEAT** this for **each TUBING**.
3. **TEST UNIT**.
4. **REMOUNT** delivery system.