

# A014-007

## Removal and Installation of Routing Valve

### Removal

- A. Remove the four Phillip screws holding the delivery system to wooden top.
- B. Remove one tubing at a time from routing valve so as not to mix them up.
- C. Remove the tubing's by pulling and twisting on the plastic sleeve clamps holding tubing to barb.
- D. Carefully remove tubing by cutting away from barb.

### Installation

- A. Slide the plastic sleeve clamps up the tubing and out of the way.
- B. Attach tubing to each barb on new part.
- C. Slide plastic sleeve clamp down over tubing and barb.
- D. Test unit before remounting delivery system with four screws to the top.