

# A10-1232

## Scaler Button Assembly Instructions

### Removal

- 1) Depressurize the dental unit.
- 2) Remove the 4 Philips screws that hold the housing for the scaler button to the base of the foot control.
- 3) Once the housing is removed, take the tubing off of the button assembly one at a time. Remove the tubing by sliding back the white plastic sleeve followed by the tubing.
- 4) Trim 1/16 off of each tube to make for a nice new connection.

### INSTALLATION

- 1) Start by installing the tubing onto the new valve by sliding a plastic sleeve onto the tubing first then push the tubing onto the metal barb followed by the plastic sleeve. (Alcohol or soapy water on the metal nipple helps these slide on easier) Grey tubing on the bottom/white tubing on the top.
- 2) Slide housing over button assembly and rotate so screw holes line up.
- 3) Install all 4 screws back into housing.
- 4) Test unit.