

A10-8941

Water Bottle Quick Disconnect Installation Instructions

1. If this unit did not previously have a quick disconnect on it remove the blue pick-up tube and replace it with the furnished longer tube. The kit contains tubes for the 1-liter and 2-liter bottle.

2. Replace the gasket in the manifold with the new gasket to insure a leak free system. Apply thread lock material to two full threads on converter. Thread converter into manifold. Tighten converter into manifold using the bottle adapter.

Note-Allow thread lock material to set for one hour.

3. Apply thread lock material to the top two threads on the bottle and then thread the adapter onto the bottle until bottomed on the gasket.


Note-Allow thread lock material to set for one hour.

4. Install bottle with adapter to converter element.

5. Turn on air pressure. Check for air leaks.

6. Turn off air pressure and remove bottle.

7. Fill bottle with water and re-install. Turn on air pressure and verify water flow is normal.

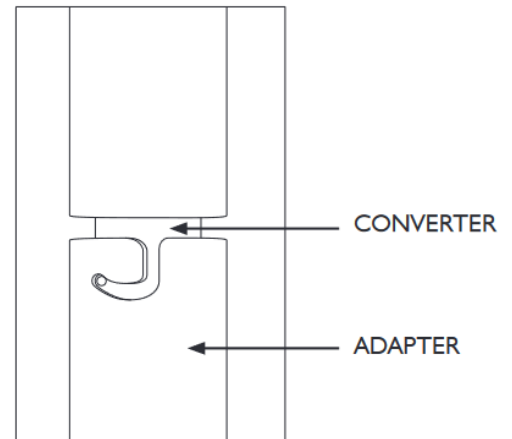

OPERATING INSTRUCTIONS

To refill bottles, simply follow these few steps:

1. Grip bottle securely and twist slowly. You will hear the air pressure automatically releasing. Pause to allow the pressure to bleed off. Continue to twist in a continuous motion to remove bottle.

2. Install a filled bottle onto the locking pins and twist to the approximate locked position.

3. Turn on the air supply to the bottle. The air pressure will move the bottle into the locked position automatically.