

# A113-403

## DENTALAIRE SYRINGE REBUILD INSTRUCTIONS

### ROUND HEADED SYRINGE

1. Depressurize Dental Machine
2. Remove pin from side of syringe housing
3. Remove buttons and springs.
4. Clean button hole with alcohol and - tip to remove deposit.
5. Insert new springs smaller side up.
6. Install new buttons by pushing into assembly all the way.
7. Replace pin into side of housing.

### SQUARE HEADED SYRINGE

1. Depressurize Dental Machine
2. Remove pin from the side of syringe housing. Use a paper Clip or small Allen wrench.
3. Remove both buttons by pulling straight out.
4. Remove both valve cores by unscrewing with supplied tool.
5. Clean both button holes with alcohol and Q-tip to remove deposit.
6. Install new valve cores with valve core tool. tighten snugly.
1. Install new buttons. Replace pin on side of housing.

