

# A120-000

## Removal and Installation of On/Off Toggle Valve

### Removal

1. (**Ultimate**) Mobile cart- remove the four Phillips screws that hold the delivery system to the dental top. There are two screws on the bottom of each side. The screw heads face the ground. (**Countertop units**)- remove the two Allen screws in the middle of each side of the delivery system. (**Wall mounts units**)- remove the four screws that secure the tray to the top of the delivery system.
2. Locate the on off switch on the front face plate on the delivery system. Unscrew the lock nut on front of the switch.
3. Pull the on/off switch through the hole of the face plate.
4. Cut each tubing as close as possible to the on/off switch. Cut the tubing's and reattach to the new on/off switch one at a time as not to mix the orientation of them up. (See #3 on installation)

### Installation

1. Unscrew the first lock nut on the new on/off switch.
2. Push the new on/off switch through the hole on the back side of the face plate then tighten down the lock nut.
3. Slide on a plastic sleeve clamp onto the cut tubing.
4. Push the tubing onto the metal bard followed by sliding the plastic sleeve clamp over the tubing and the barb so the tubing is secured. (If needed lubricate the tubing and barb with soapy water or alcohol)
5. Repeat for the other tubing connection.
6. Test and remount the delivery system.

