

A126-025

Foot Control Repair Kit

Removal

1. First depressurize the dental machine.
2. Remove the four small Phillip screws on the bottom of the outer ring of the foot pedal.
3. Remove the metal ring, plastic cover and spring from the top of the foot pedal.
4. Remove the two screws on the bottom center of the foot pedal that hold the body to the base plate.
5. Remove the O-ring from the bottom of the body along with a small spring and poppet.
6. Now with the body removed take a small instrument and remove the stem by pushing the stem out from the bottom through the top.

Installation

1. Clean baseplate and body.
2. Place supplied O-ring in groove on bottom of the body.
3. Insert poppet into body followed by the small spring with the smaller end facing the poppet.
4. Screw down body evenly to base plate.
5. Insert stem into body.
6. Place large spring on top of body.
7. Align cover and ring.
8. Start all four screw's that hold the ring to the base plate, once all four are started tighten them evenly until all four are tight.
9. Pressurize unit and test.

