

A23-000

Water Relay Instructions for the High Speed Drill

Removal

1. **(Ultimate)** Mobile cart- remove the four Phillips screws that hold the delivery system to the dental top. There are two screws on the bottom of each side. The screw heads face the ground. **(Prestige)** Mobile cart- Flip up the lid of the unit and remove the plastic storage tray to access the water relay. **Countertop units-** remove the two Allen screws in the middle of each side of the delivery system. **Wall mounts units-** remove the four screws that secure the tray to the top of the delivery system.
2. Locate the water relay for the drill. The middle tubing is the water out to the hand piece.
3. Cut the tubings as close as possible to the water relay. Only cut one tubing at a time then reattach the tubing to the new water relay one tubing at a time as to not mix them up.

Installation

4. Slide on a plastic sleeve clamp onto each of the cut tubings.

Push on each of the tubings to the correct barbs followed by sliding on the plastic sleeve clamps over the tubing and barb so it has a secure connection. The tubing only needs to be pushed onto the barb about $\frac{3}{4}$ of the way. Soapy water or alcohol helps the tubing and sleeves slide on easier.

