

# PART # A23-001

## Removal and installation of the scaler Water Relay

**NOTE** THIS NEW VALVE MAY LOOK DIFFERENT FROM THE ONE IN YOUR UNIT.

### Removal

- **(Ultimate) Mobile cart**- remove the four Phillips screws that hold the delivery system to the laminated top. There are two screws on the bottom of each side. The screw heads face the ground. **(Prestige) cart**- Flip up the lid of the unit and remove the plastic storage tray to access the water relay. **Countertop** units- remove the two Allen screws in the middle of each side of the delivery system. **Wall mounts**- remove the four screws that secure the tray to the top of the delivery system.
- Locate the water relay for the scaler by following the water line that comes from the scaler tubing.
- Cut each tubing off of the old valve as close as you can to the valve as to leave as much extra tubing as possible for the new valve.

### Installation

- How to connect tubing's to the valve. Slide new plastic sleeves onto tubing before the tubing's are slid onto the barb of the new valve. An all-purpose soapy spray or alcohol will help the tubing and sleeves slide onto their connections.
- The smaller grey tube with the tee attached to it will hook to the barb labeled air on the picture below.
- The clear line coming from the scaler will connect to the barb on the side of the water relay; this line is labeled in picture below as # 1.
- The last grey water line will connect to the barb on the opposite side from the air; this line is labeled in the picture below as # 2.
- Once the install is complete, test the unit and remount delivery system.

Air (grey tube with 3 way Tee)



Water 1 (clear scaler tubing)

