

# A26-000

## Standard Foot Pedal Installation Instructions

### Removal

1. Remove the four Phillips screw's holding the delivery system to the top.
2. Remove and install one foot control tubing at a time, so as not to mix them up. (Note one of the larger tubing's has a ridge on it to indicate which large tube is the main air supply).
3. Remove the tubing by pulling and twisting on the plastic sleeve clamp then cut the tubing from the barb.

### Install

1. Slide a plastic sleeve clamp up the tubing, and then push the tubing onto the barb followed by the sleeve. (Note soapy water or alcohol helps the tubing and sleeves slide on easier).
2. Repeat for each tubing.
3. Test unit.
4. Remount delivery system to the top.