

A26-320

Foot Pedal W/Scaler Button Installation Instructions

Removal

1. To gain access to the tubing connections- (**Ultimate**) Mobile cart- remove the four Phillips screws that hold the delivery system to the top. There are two screws on the bottom of each side. The screw heads face the ground. (**Prestige**) cart- Flip up the lid of the unit and remove the plastic storage tray to access the inside. **Countertop** units- remove the two Allen screws in the middle of each side of the delivery system. **Wall mounts**- remove the four screws that secure the tray to the top of the delivery system.
2. Remove and install one foot control tubing at a time, so as not to mix them up. (Note one of the larger tubing's has a ridge on it to indicate which large tube is the main air supply).
3. Remove the tubing by pulling and twisting on the plastic sleeve clamp. Then cut the tubing from the barb.

Installation

1. First separate the new tubing's and make each tube the same length as the old tubing that was removed. Slide a plastic sleeve clamp up the tubing, and then push the tubing onto the barb followed by the sleeve. (Note one of the larger tubing's has a ridge on it to indicate which large tube is the main air supply also soapy water or alcohol helps the tubing and sleeves slide on easier). Note: Use supplied plastic Tee for the smaller tubing if installed one is damaged while switching foot controls.
2. Repeat for each tubing.
3. Test unit and re-mount delivery system.