

A28-018

Removal and Installation of Inline Mini Regulator

1) Removal

A) Depressurize the dental machine.

B) Locating the inline mini regulator. **(Ultimate) Mobile cart-** remove the four Phillips screws that hold the delivery system to the top. There are two screws on the bottom of each side. The screw heads face the ground. Follow the small grey line from the water bottles connection until it meets up to the inline mini regulator.

(Prestige) Mobile cart- Flip up the lid of the unit and remove the plastic tray to access the inside of the unit. Locate the mini regulator by following the small clear line from the water bottles up inside of the unit. (Note the new part might look different from the old part).

c) Cut the tubing as close to the valve as possible on both sides.

2) Installation

A) Have the arrow on the inline mini regulator point towards the water bottles.

B) Slide a plastic sleeve clamp onto both of the tubing that were cut from the old valve.

c) Push on each of the tubing's to the correct barbs followed by sliding on the plastic sleeve clamps over the tubing and barb so it has a secure connection. (If needed lubricate the tubing and barb with soapy water or alcohol to help them slide on easier.)

D) Pressurize machine and test.