

C0124

Valve Safety 8 Bar M1/4

Important- This valve is not for draining the air out of the compressor. This is a safety valve, which is not to be touched.

Removal

1. Make sure to completely depressurize the compressor.
2. Remove old valve by unscrewing counter clock wise with a wrench.

Installation

3. Install using thread sealer or Teflon tape (plumbers tape) to seal the connection.
4. Thread the valve in with your fingers first then tighten the valve with a wrench. Note the valve will not go flush into the piece it screws into.
5. Once it is tight go ahead and run the unit and spray with soapy water to detect a leak if needed.

Part # C0124