

DA400LED LG TURB Turbine Replacement Instructions

Step 1



Unscrew back push button cap from drill.
Use cap tool, that is included with handpiece, to unscrew cap.

Cap Tool Item Number:
DTPLED-CPT

Step 2



Remove turbine carefully by pushing on bur to push it out.
Identify your turbine color and follow the corresponding steps.
Gold Turbine: If removing gold colored turbine then remove wavy washer.
Silver Turbine: If removing a silver turbine, cap will have 1 flat metal washer and head of the drill will have 2 wavy washers.

Step 3



Install new turbine by putting flat metal washer into the cap. Then, insert new turbine into cap.

Bearing on the back of turbine will need to slide into an o-ring in cap. Turbine must be perfectly straight to slide into o-ring. Pushing on the button will help the turbine slide into o-ring.

Step 4




Stack rubber washer on top of 2 wavy washers. Rubber washer's concaved side must be facing down.

One wavy washer will be the original from the handpiece and the other washer will be the one that came with the new replacement turbine.

NOTE: If replacing a replacement turbine with another replacement turbine, use one existing washer and one new wavy washer.


Step 5

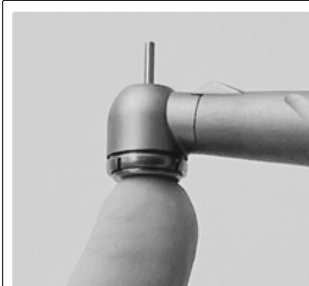

Insert turbine and cap into head of drill perfectly straight.

The bearing on the front of turbine will also need to slide into an o-ring in head of the drill. While inserting the turbine, push on button to get the turbine to slide into the o-ring.

Step 6

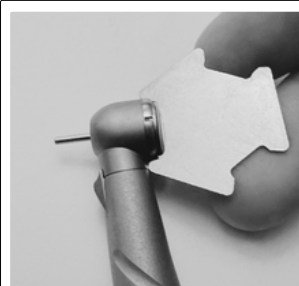

Tighten cap finger tight. Then, use cap tool.

Once turbine slides in, tighten cap using your fingers. If the bur spins freely, then use cap tool to tighten the cap.

Need more help?



(800) 866-6881



customerservice@dentalaire.com