

LED Swivel High-speed Handpiece

Quick Start Guide


DA400LEDMH





How to Lubricate the Handpiece

Lubricate handpiece right before every dental.

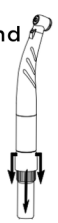- 1** Unscrew handpiece and remove from tubing.



- 2** Locate ports on the bottom of handpiece. **Midwest 4-Holes**


- 3** Spray generous amount of lubricant into the smaller of the 2 larger ports.


- 4** Connect and screw back handpiece onto tubing.
- 5** Run handpiece for 20-30 seconds. This will help run the lubricant through the turbine.

How to Insert Bur

- 1** Press end cap button firmly and push bur in until it stops.

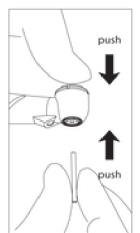




ALWAYS put bur all the way in.
NEVER put bur in just half way.

How to Remove Bur

- 1** Firmly press end cap button and pull out bur.

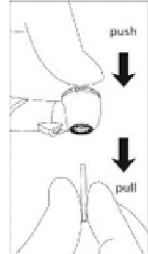



DO NOT run the high-speed handpiece without a bur in it. This will damage the chuck.

How to Clean Handpiece

1. Following your country specific directives, standard and guidelines Steam sterilization class B (pre and post vacuum) with sterilizers in accordance with EN 13060. Sterilization holding time a minimum of 3 minutes at 134°C (273° F)

OR

2. Steam sterilization class S with sterilizers in accordance with EN 13060. The sterilizer manufacturer must give the express approval for the sterilization of turbine handpieces. Sterilization holding time a minimum of 3 minutes at 134°C (273° F)

Need more help?

 (800) 866-6881  info@dentalaire.com

PARTS AND ACCESSORIES

DA100JP

All-In-One Cleaner and Lubricant

DA400MHTURB

Turbine for DA400LEDMH