

DA400MHTURB Turbine Replacement Instructions

Step 1



Unscrew push button cap from drill.
Use cap tool, that is included with handpiece, to unscrew cap.

Cap Tool Item Number:
DTPLED-CPT

Step 2




Remove turbine carefully.

Drill will have a small metal washer in the head and a wave washer in the cap.

NOTE: Make sure the washers stay in place.

Step 3

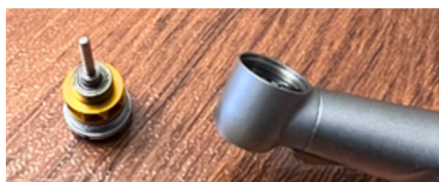

Insert new turbine into cap, first.

Bearing on back of turbine needs to slide into an o-ring in cap. Pushing on the button can help the turbine slide into the o-ring.

NOTE: Turbine must be perfectly straight to slide into o-ring.

Step 4



Insert turbine and cap into head of drill perfectly straight.

Bearing on front of the turbine also needs to slide into an o-ring in the head of the drill. While inserting the turbine, push on the button to get turbine to slide into o-ring.

Step 5



Tighten cap finger tight. Then, use cap tool.

Once turbine slides in, tighten cap using your fingers. If the bur spins freely, then use cap tool to tighten the cap.



NOTE: Original turbine is gold in color.
New replacement turbine is silver.

Need more help?

 (800) 866-6881  customerservice@dentalaire.com