




# Low-speed Handpiece Quick Start Guide

DA500A







## How to Lubricate the Handpiece

*Lubricate handpiece once at the start of the day.*

- 1** Unscrew handpiece and remove from tubing.
 
- 2** Locate ports on the bottom of handpiece.
 
 Midwest 4-Holes
- 3** Spray generous amount of lubricant into the smaller of the 2 larger ports.
 
- 4** Connect and screw back handpiece onto tubing.
- 5** Run handpiece for 20-30 seconds. This will help run the lubricant through the motor.

## How to Insert & Remove Bur or Angle Attachment

- 1** Twist chuck locking and releasing ring to left to unlock position.
 
- 2** Insert bur or angle attachment.
 
- 3** Twist chuck locking and releasing ring right to locked position.
 
- 4** To remove bur turn ring left to unlocked position.
 



**DO NOT** close the chuck (locked position) unless a bur or angle attachment is installed.

## Rotational Change



**Forward rotation** = Twist rotational control ring index (screw) right to full forward position

**Reverse rotation** = Twist rotational control ring index (screw) left to full reverse position

**No rotation** = Place rotational control ring index (screw) in between reverse and forward rotation

## How to Sterilize Handpiece

1. Ensure that the attachment has been removed from the motor.
2. Clean the exterior with a damp towel to remove any debris or paste.
3. Place handpiece in appropriate sterilization bag, as recommended by sterilizer manufacturer.
4. Sterilize by steam autoclave at 135°C (275° F) for 25 minutes.
5. Remove the sterilization bag from the sterilizer immediately after cycle is completed and allow the components to cool in the bag.
6. Store the components in the sealed sterilization bag until ready for use.

## Need more help?



(800) 866-6881



info@dentalaire.com

## PARTS AND ACCESSORIES

DA100JP

All-In-One Cleaner and Lubricant