

Low-speed Handpiece Quick Start Guide

DA700A



Handpiece Maintenance

Perform cleaning procedure routinely on a monthly basis, or if motor becomes sluggish.

- 1** Disconnect the air tubing connector.
 
- 2** Locate ports on the bottom of handpiece.
 
- 3** Apply 3 to 4 drops of **Air Motor Cleaner** into small inlet tube on handpiece connector.
 
- 4** Reconnect the air tubing connector.
 
- 5** Run the motor in forward and reverse for 10 seconds or until the motor runs smoothly.
 

How to Insert & Remove Bur or Angle Attachment

- 1** Twist chuck locking and releasing ring to left to unlock position.
 
- 2** Insert bur or angle attachment.
 
- 3** Twist chuck locking and releasing ring right to locked position.
 
- 4** To remove bur turn ring left to unlocked position.
 



DO NOT close the chuck (locked position) unless a bur or angle attachment is installed.

Rotational Change



Forward rotation = Turn the ring clockwise (towards the "F" for forward rotation).

Reverse rotation = Turn the ring counter clockwise (towards the "R") for reverse rotation.

No rotation = Aligning the mark on the ring with the "O" in the center will stop the motor completely.

How to Sterilize Handpiece

1. Ensure that the attachment has been removed from the motor.
2. Clean the exterior with a damp towelette or lavage to remove any debris or paste.
3. Place handpiece in appropriate sterilization bag as recommended by sterilizer manufacturer.
4. Sterilize by steam autoclave at 275 degrees F (135 degrees C) for 25 minutes.
5. Remove the sterilization bag from the sterilizer immediately after cycle is completed and allow the components to cool in the bag.
6. Store the components in the sealed sterilization bag until ready for use.

Need more help?



(800) 866-6881



info@dentalaire.com

PARTS AND ACCESSORIES

DA100MC

Air Motor Cleaner

DA500SP

Slow Speed Shipping Post