



## Canister Turbine Installation Instructions

DANSK-A485  
DA400RCA  
DA400RC

### Removal

#### Step 1



##### Remove back cap.

Use cap tool to remove back cap from handpiece.

Using pliers can cause damage to the cap and head of handpiece.

#### Step 2




##### Remove existing turbine.

Push the bur against a hard surface, like a table, to push turbine out of the head of the drill.

### Installation

#### Step 1

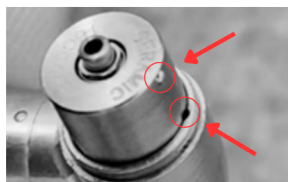

##### Install turbine.

Locate the small alignment ball on the upper turbine surface, opposite side of the two holes. Then, install by sliding this ball into the handpiece canister's slot.

#### Step 2



##### Re-install cap.

Use cap tool to re-install back cap from handpiece.

Using pliers can cause damage to the cap and head of handpiece.

### Maintenance

- Lubricate handpiece before each use
- Use alcohol to clean handpiece
- Do not use any chemicals or chlorine
- Do not put handpiece in an ultrasonic cleaner
- Follow instructions in owner's manual for autoclave instructions



Scan to watch  
tutorial video

#### How to Lubricate High-speed Handpiece

##### Step 1

Unscrew handpiece and remove from tubing.



##### Step 2

Locate ports towards the back end tubing.



##### Step 3

Spray generous amount of lubricant into the smaller of the 2 larger ports.



##### Step 4

Connect and screw back handpiece onto tubing.

##### Step 5


Run handpiece for 20-30 seconds. This will help run the lubricant through the turbine.



##### CAUTION

Failure to lubricate handpiece before/after use will void the warranty.

### Helpful Parts to Remember

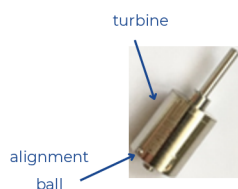

internal slot  
opening in  
handpiece

All-in-One Cleaner  
and Lubricant  
Part #: DA100JP



Need more help?



(800) 866-6881



customerservice@dentalaire.com