



Product Description

PetSoft Pro™ is a latex-free, handheld retractor designed for use during any chairside procedure.

- Resembles a dental mirror in form
- Flexible neck and protective shield for effective tissue retraction
- Autoclavable and reusable
- Suitable for a variety of tissue protection and retraction needs

Intended Use

PetSoft Pro is designed for any tissue protection or retraction procedure and is autoclavable for up to 30 cycles.

Directions

Supplied non-sterile.

- Remove instrument from the package.
- Bend neck to adjust desired position for specific application and begin retracting.

Cleaning Instructions

- Wash with an enzyme-based or similar detergent prior to sterilization.
- Before sterilizing, make sure instruments are completely dry. Inspect instrument for any damage.
- Steam autoclave/sterilize at 250°F to 273°F (121°C to 134°C) for 20 minutes. Follow your autoclave manufacturer's instructions regarding drying time/cycle. (Autoclavable up to 30 cycles)
- After sterilization, the devices are to remain in their sealed package and open only at point of use.

! Bendable neck will weaken after repeated use.

! Over time and after repeated autoclaving, PetSoft Pro will begin to show signs of normal wear. Visual changes like wear and tear or deterioration will indicate that the product should be replaced.



DentalAire Item Number	Description
DTP10972	PetSoft Pro (Autoclavable), 12 Pack for Professional

Questions?



(800) 866-6881



info@dentalaire.com