



DENTALAIRE

Meeting and Exceeding the Needs of Veterinary Dentistry since 1988

Instruments

SimpliScale Periodontal Therapy

Unique instruments designed for treating Veterinary Patients



Features

- Ergonomic non-slip grip
- Superior tactile sensitivity
- Extremely wear resistant DuraGradeMax steel with light weight special alloy core
- Exclusive universal curette design
- Highest grade steel
- Autoclavable instruments and cassette
- Non-fade instrument colors
- Color-coded handles

Benefits

- Reduces finger fatigue
- Allows precise treatment
- Extends instrument life and decreases weight
- Three universal curettes replace six normal curettes
- Extends the time between sharpening
- Improved patient safety
- Instruments look new for years
- Serve as guide for correct & safe use

www.Dentalaire.com | 800.866.6881

Dentalaire | 17742 Mitchell North, Suite B Irvine CA 92614

info@dentalaire.com



DENTALAIRE


Meeting and Exceeding the Needs of Veterinary Dentistry since 1988

Instruments


SimpliScale Periodontal Therapy

Unique instruments designed for treating Veterinary Patients

The Dentalaire SimpliScale Periodontal kit has been designed to address all periodontal needs for the clinician. Utilizing the unique universal Gracey design removes the need to change Gracey instruments while performing lingual and buccal cleaning techniques. Each universal Gracey instrument replaces two standard Gracey instruments.

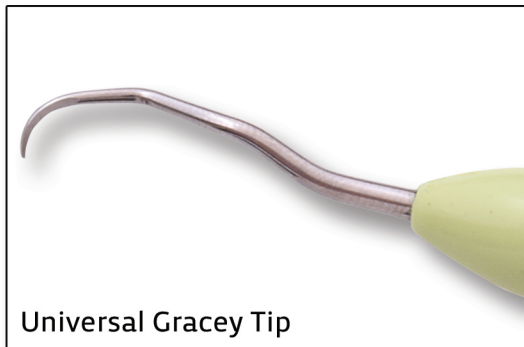

Universal Gracey Tip

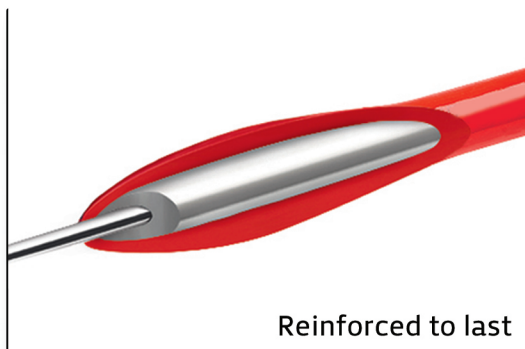

Reinforced to last

DTP11295 SimpliScale Periodontal Therapy

Includes 7 Instruments / Cassette:

- DTP11291 - SimpliScale Probe/Explorer (Blue)
- DTP11292 - SimpliScale Universal Gracey Petite (Green)
- DTP11293 - SimpliScale Universal Gracey Anterior (Green)
- DTP11294 - SimpliScale Universal Gracey (Green)
- DTP11296 - SimpliScale Scaler Posterior Petite (Red)
- DTP11297 - SimpliScale Scaler Anterior (Red)
- DTP11298 - SimpliScale Scaler Posterior (Red)
- DTP11299 - SimpliScale Cassette

- **Probe Explorer:** Used to find pathology both above and under the gumline.
- **Universal Gracey Petite:** The groundbreaking design replaces two standard Gracey Curettes and is used under the gumline in teeth behind the canine teeth. One end of the instrument is used to access treatment sites from the buccal (outside) of the dental arcade and one end from the lingual/palatal (inside) of the dental arcade. The smaller petite blade of this instrument fits into narrower spaces between teeth and periodontal pockets on smaller teeth.
- **Universal Gracey Anterior:** Replaces two regular Gracey curettes. Used for incisors and canine teeth. The smaller blade on one end is used under the gumline in smaller teeth and the larger blade on the other end is used under the gumline in larger teeth.
- **Universal Gracey:** The groundbreaking design replaces two standard Gracey curettes and is used underneath the gumline in teeth behind the canine teeth. One end of the instrument is used to access treatment sites from the buccal (outside) of the dental arcade and one end from the lingual/palatal (inside) of the dental arcade. The larger blade size of this instrument is used for wider spaces between the teeth and periodontal pockets on larger teeth.
- **Scaler Posterior Petite:** Used to clean the crowns of smaller teeth behind the canine teeth. One end of the instrument is used to access treatment sites from the buccal (outside) of the dental arcade and one end from the lingual/palatal (inside) of the dental arcade.
- **Scaler Anterior:** Used to clean the crowns of the canine teeth and incisors. The larger blade on one end is used larger teeth and the smaller blade on the other end is used for smaller teeth.
- **Scaler Posterior:** Used to clean the crowns of larger teeth behind the canine teeth. One end of the instrument is used to access treatment sites from the buccal (outside) of the dental arcade and one end from the lingual/palatal (inside) of the dental arcade.

The exclusive Universal/Area specific design of the Universal Gracey curettes simplify treatment, replacing two normal Gracey curettes with a single instrument.

www.Dentalaire.com | 800.866.6881

Dentalaire | 17742 Mitchell North, Suite B Irvine CA 92614

info@dentalaire.com